
NEW
ReTreaX Ionic
CONTEMPORARY MASSAGE CHAIR

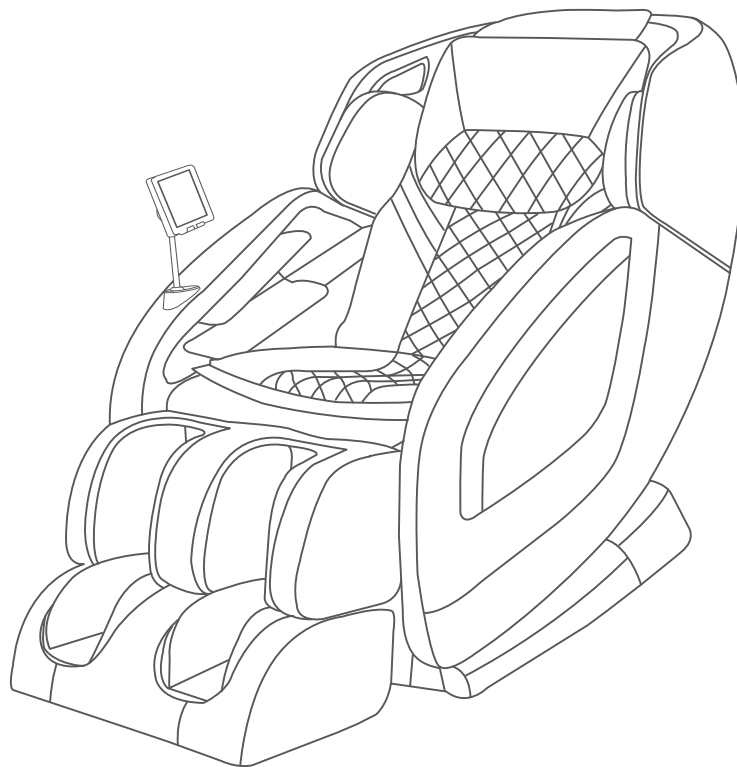Rev.1.0

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CONTENTS

Safety Instructions	03
Warranty Claims	05
Product Parts	06
Moving the Chair	07
Controller Diagram	08
Product Operation Guide	09
Product Features	14
Cleaning And Maintenance	15
Questions And Answers(Q&A)	16
Troubleshooting	17
Product Specification	19
Contact Us	36

Safety Instructions

Warning!

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in the risk of electric shock.

Check with a qualified electrician and service personnel if you are in doubt as to whether the product is properly grounded. Do not modify the product's accompanying plug. If the outlet is not suitable, be sure to have a proper outlet installed by a qualified electrician.

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire, or injury.

Read the following details before use:

1. Be sure to insert the massage chair plug into the proper electrical outlet.
2. Make sure that the power plug is securely inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use and before cleaning.
4. DO NOT use the massage chair in places where the humidity is high.
5. DO NOT use the massage chair outdoors and avoid places with direct sunlight.
6. If the power goes off during use, be sure to turn the power switch to the OFF position and pull the power plug out from the socket to prevent any damages when the power comes back.
7. Never operate the massage chair if the power cord is wet or damaged.
8. Never plug/unplug the power cord from the socket with wet hands.
9. Close and continuous supervision is necessary when this appliance is used by or near children, invalids or disabled persons.
10. Use this appliance only as described in this user manual.
11. Do not use any attachments other than those recommended by OGAWA.
12. Do not stand, sit or jump in the calves rest when the massage chair is in use. This may result in an accident or injury.
13. Do not attempt to stand on the backrest and armrest .This may result in an accident or injury.
14. Be careful not to fall asleep during the massage.

Safety Instructions



15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair.
16. Do not remove the massage chair by pulling the power supply cord.
17. Be sure to check that there is no one (children in particular), pets or obstacles under the massage chair prior to lowering the leg rest /backrest.
18. Always check whether the surface of the massage chair is torn or ripped before use. If you find any unfamiliar tears in any area of the cloth in the backrest, do not use the massage chair. Kindly contact OGAWA service center immediately for repairs.
19. This appliance is designed for household use only.
20. For those suffering from illnesses, please consult your doctor on whether you can use the massage chair if in doubt.
21. The following persons are requested to consult a doctor prior to use in order to reduce the risk of the user feeling sick or suffering from an injury:
 - Those who have a malignant tumor;
 - Those who suffer from heart disease;
 - Those who are experiencing pregnancy, menstruation and puerperal;
 - Those who require complete rest;
 - Those who are under medical treatment;
 - Those who suffer from back problems or whose backbone is abnormal or bent.
22. If you experience pain during use, stop the massage chair immediately. If any injuries happen, consult a doctor immediately. Never massage on swollen or inflamed parts of the body.
23. This product is a non-professional appliance designed to provide a comfortable massage. Do not use this product as a substitute for medical treatment.
24. Do not use this massage chair for more than 30 minutes at one time and kindly limit the massage to a concentrated area for not more than 15 minutes. Over-massaging can result in overheating, and thus reducing the product's lifespan.
25. Do not operate where aerosol (spray) products are used or where oxygen is being administered.
26. Children shall not play with the appliance.
Cleaning and user maintenance shall not be made by children without supervision.

27. If the power plug is damaged, it must be replaced by a special plug available from OGAWA or authorized agents.
28. As a safety precaution , Motor will stop working automatically (for protection) if apply too strong external force during use. Once appear above, switch OFF power ,and switch ON again after 10 seconds . Please pay attention to this ,especially for user whose weight over 100KG.
29. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
30. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved

Warranty Claims

1. Warranty is not transferable.
2. This warranty is effective only if the product is purchased from OGAWA.
3. Proof of purchase (original receipt) is required for all warranty repairs.
4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited to the duration stated in the warranty policy.
5. Damage due to misuse, improper treatment and unauthorized modification and repair are not covered by this warranty.
6. Warranty does not cover accessories and attachments which do not belong to the massage unit.
7. Warranty is not effective to rental, business, commercial, institutional or other non-residential users.
8. All services covered by this warranty must be approved by OGAWA and repaired by authorized technicians only.
9. If replacement parts for defective materials are not available, OGAWA reserves the right to make substitution in lieu of repair or replacement.



Product Parts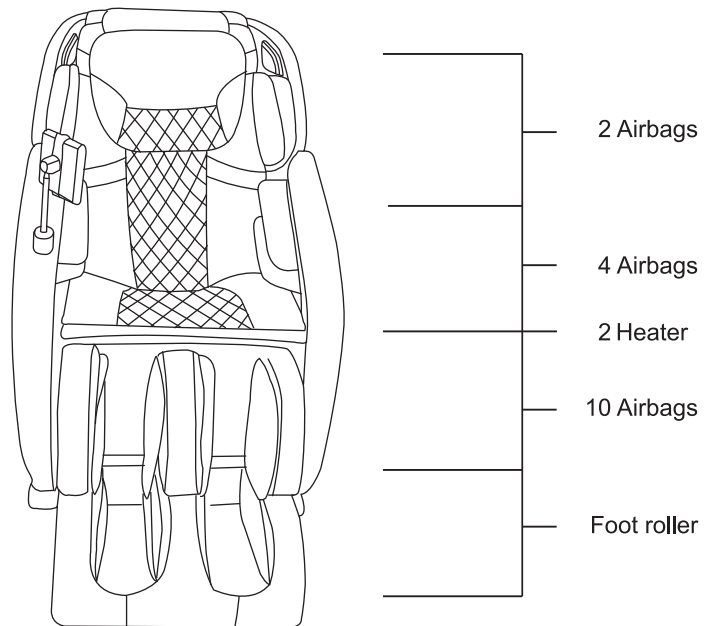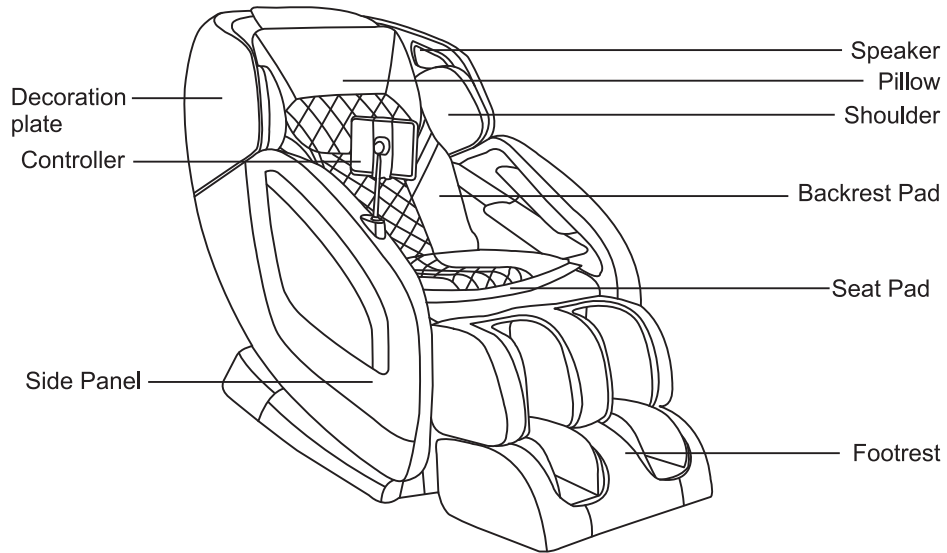

Important-product image or content descriptions may slightly differ due to product quality improvement.

Moving the Chair


Press down the backrest, before reclining the chair to approximate 45° to easily move the chair around the house with a slight pull or push.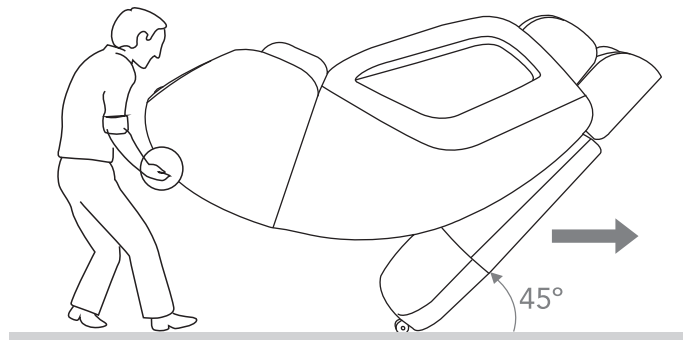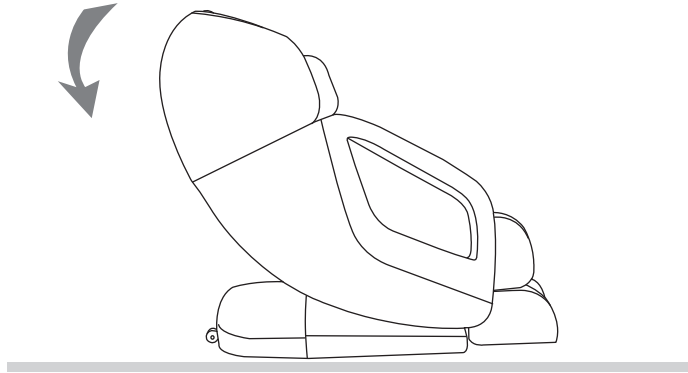

Caution:

- i. Before moving the chair, please put the footrest back in the original position. Otherwise, injuries might occur in the process of moving the chair.
- ii. Avoid moving the massage chair when there is someone seated on it.
- iii. Placing a mat underneath the massage chair is advisable, as the moving process might leave scratches on the floor surface.

Controller Diagram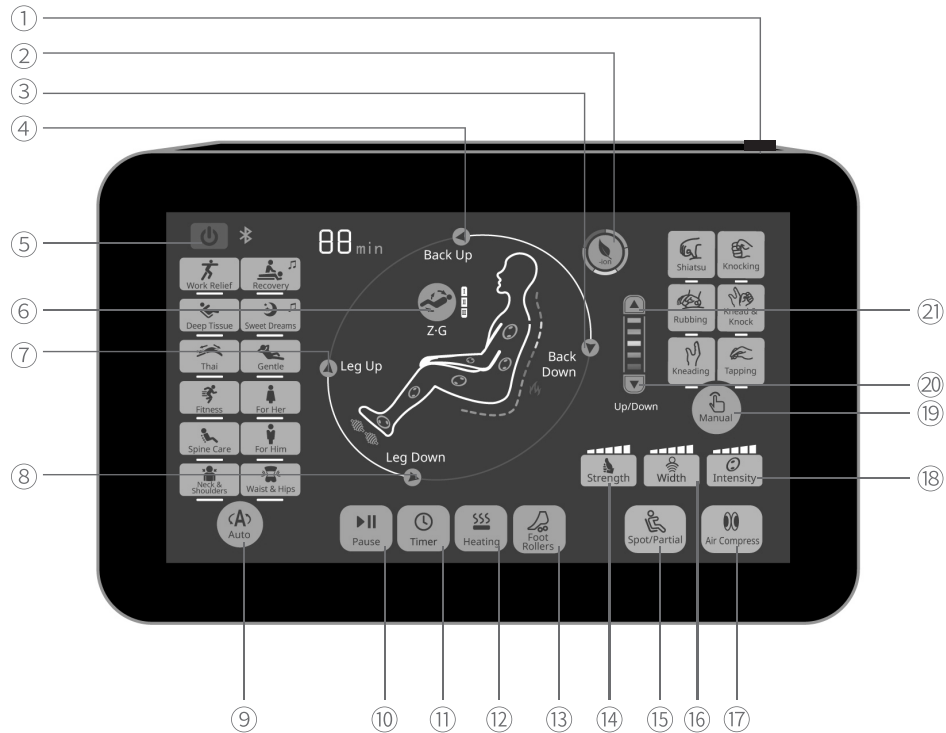

Controller

- | | | |
|------------------------|-----------------------|----------------------------|
| ① Power Switch | ⑧ Legrest Descend | ⑮ Spot/Partial |
| ② Negative Ion | ⑨ Auto Programs | ⑯ Width Adjustment |
| ③ Back Cushion Descend | ⑩ Pause | ⑰ Air Compress Massage |
| ④ Back Cushion Ascend | ⑪ Timer | ⑱ Air Intensity Adjustment |
| ⑤ On/Off | ⑫ Heating Function | ⑲ Manual Massage |
| ⑥ Zero-Gravity | ⑬ Foot Roller Massage | ⑳ Massage Roller Down |
| ⑦ Legrest Ascend | ⑭ Strength Adjustment | ㉑ Massage Roller Up |


Instructions of buttons

! Warning: Only rated supply 220V must be used to avoid damage to electronic elements.

Product Operation Guide

1. Powering up the massage chair

Please make sure that the power switch at the back of the massage chair is in the off "O" position, then insert the power cable into the port to the left of the switch. Once firmly plugged in, proceed to plug the power cord into the outlet. Then put the switch to the on "I" and the remote should light up.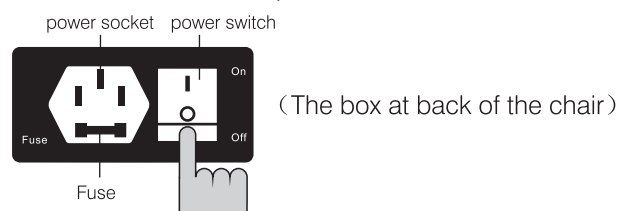

2. Starting your massage

Sit in the chair with your buttocks firmly pressed against the backrest to ensure that the chair will provide a consistent massage pressure from top to bottom. The press "on" button and select a massage.


Press Time Setting button to adjust massage time. Maximum is 30 minutes.

3. Adjusting the angle of the recline

Please adjust the backrest and legrest angle prior to selecting a massage function. Getting into position before selecting the massage function allows for the chair to deliver a more accurate massage. The chair and legrest can be adjusted simultaneously or independently, depending on the button selected on the remote as illustrated below.

Press the " " button to start zero gravity massage.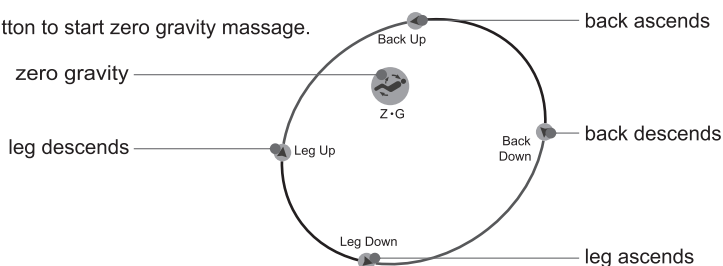

4. Stop or finish the massage

Normal stop : the timing function is pre-set 15 minutes. At time out the program will stop and keep the setting posture.

Force stop: when press the restore during massage, it will stop massage immediately.


Press button again to continue.

Product Operation Guide

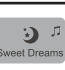5. Selecting a preset programmed massage

Automatic programs combine shiatsu, kneading, tapping and air massage.

Button	Description	Main function
	 Work Relief massage program	Press 'Auto' button one times to start Work Relief massage. * This is an exclusive massage program for the workplace, it can ease muscles and recuperate body.
	 Deep Tissue massage program	Press 'Auto' button two times to start Deep Tissue massage. * A full body deep massage includes shiatsu, knocking and tapping to ease and relax muscles after intense exercise.
	 Thai massage program	Press 'Auto' button three times to start Thai massage. * This massage inspired from Thai massage, the strong kneading and stretch of the body can effectively stretch legs.
	 Fitness massage program	Press 'Auto' button four times to start Fitness massage. * Promote the blood circulation system to improve the muscle recovery after exercise.
	 Spine Care massage program	Press 'Auto' button five times to start Spine Care massage. * Mainly focus on your full back with a deep massage that includes air compression to relieve the lumbar spine pain.
	 Neck & Shoulder massage program	Press 'Auto' button six times to start Neck & Shoulder massage. * Mainly focus on your neck and shoulders with a slow massage that incorporates soothing heat to relieve neck and back pain.
	 Recovery massage program	Press 'Auto' button seven times to start Recovery massage. * A full body deep massage includes shiatsu, knocking and tapping to ease and relax muscles. (Come with soothing music)
	 Sweet Dreams massage program	Press 'Auto' button eight times to start Sweet Dreams massage. * A mild massage that is great for deep relaxation. The program help improve the quality of sleep. (Come with soothing music)
	 Gentle massage program	Press 'Auto' button nine times to start Gentle massage. * A full body gentle massage includes kneading and heating, designed for daily use.
	 For Her massage program	Press 'Auto' button ten times to start For Her massage. * A full body gentle massage includes kneading and heating, specially designed for female.
	 For Him massage program	Press 'Auto' button eleven times to start For Him massage. * A full body strong massage includes kneading, specially designed for male.
	 Waist & Hip massage program	Press 'Auto' button twelve times to start Waist & Hip massage. * A full body massage, mainly focus on your lower back, waist and hips with a deep massage that includes air compression to relieve lower back pain.

※ For above programs, the preset time is 15 minutes and the default of airbag intensity is low.

Press the “ Intensity ” button to adjust the intensity of air pressure.

※ The above automatic full-body massage courses cannot be combined with other programs.

Product Operation Guide

Manual setting course

Introduction of manual setting course

Partial air massage is the course which targets different needs and areas of body.

Button	Description	Main function and adjustment guide
	 Shiatsu	Press 'Manual' button one time to start shiatsu massage. The width is adjustable.
	 Rubbing	Press 'Manual' button two times to start rubbing massage. The strength is adjustable.
	 Kneading	Press 'Manual' button three times to start kneading massage. The strength is adjustable.
	 Knocking	Press 'Manual' button four times to start knocking massage. The strength and width are adjustable.
	 Knead & Knock	Press 'Manual' button five times to start knead & knock massage. The strength and width are adjustable.
	 Tapping	Press 'Manual' button six times to start tapping massage. The strength and width are adjustable.

※ The above manual setting course could not be combined with other courses.

※ For the above manual setting course, the default setting for the massage is spot-specific with medium width. Speed and width are adjustable by pressing relative button.

Function introduction for manual setting course

Button	Description	Main function
	Strength adjustment	There are five levels to choose from.
	Width adjustment	There are three levels to choose from: narrow, medium and wide.
	Full course/Partial massage	Press the button one time to start full course massage . Press two times to start the partial massage. The third time the rollers will stay in one location.
	Message roller move up Message roller move down	Press down the button to move message roller, release the button to stop.

Product Operation Guide

Functions on buttons

Introductions on buttons used for manual setting course

Button	Description	Main function
	Air compress massage	Press one time to start air compress massage for upper body. Press two times to start air compress massage for lower body. Press three times to start air compress massage for full body. To close, press four times.
	Air compress intensity adjustment	Air compress intensity is adjustable. It could be adjusted only in air pressure massage. There are five levels to choose from.
	Foot roller massage	Foot roller massage. Press the button to start foot rolling massage, press again to stop.
	Heating function	Lower back soothing heat. Press the button to start waist heating massage, press again to stop.
	Negative ions therapy function	Negative ions therapy function Press the button to start the negative ions therapy function, press again to stop.

Music Function

Connecting your mobile device: Turn on Bluetooth of your mobile phone or tablet, then select Bluetooth device (code OG5572) . After pairing with the ReTreaX massage chair, you can now play your favourite music or podcast while enjoying massage.

Product Operation Guide

End Operation

Please press the “ON/OFF” button after use, and the chair will stop its operation, while the massage rollers will return to their original position; then switch the power supply button under the chair to “O” and unplug the machine from the electrical outlet.

⚠ Warning: Supervision and guidance are necessary when disable, people who have obstacles in sense or spiritual, who are inexperienced in using it and who are lacking of the knowledge of it use the massage chair (including children). Unless there is someone who in charge of their safety, the massage chair is unavailable for them. Please take care of your children and make sure the chair not is played by them. There is heat function for the chair and should be cautions by those who insensitivity of heat.

Product Features

The ReTreaX massage chair combines various of the latest upgrade version technologies such as negative ions, music therapy, and a 7" inch LCD touchscreen controller. It provides the right ambiance to relieve stress and worry by invoking a restful state of mind and helping melt away your troubles.



1. The comfortable yet modern design offers minimal and aesthetic appeal to fit any space smartly.
2. Negative ions therapy are not only capable of purifying the air but also beneficial for the human health and overall well-being.
3. Enjoy the music therapy while massaging helps cultivate emotional balance and calm your mind.
4. 7" inch LCD touchscreen controller enable you to control the massage chair easily at your fingertips.
5. Comes with 12 extensive auto massage program to suit your preferences at ease.
6. A comprehensive full body air compress massage on your shoulders, arms, legs & feet.
7. Lower back soothing heat therapy improve blood circulation which can help to promote healing and reduce inflammation.
8. The foot roller performs acupressure massage which applies pressure and corresponds to various acupoints on the feet.
9. 3 levels of zero gravity position beneficial for relaxation and stress relief.
10. S+L track provides a full back body massage from your neck to your hips.

Cleaning And Maintenance

Cleaning of the synthetic leather


Wipe the synthetic leather with a soft dry cloth.

- **Do not clean the product with chemicals such as thinners, gasoline and alcohol.**
- **When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.**
- **If the synthetic leather is very dirty, wipe it with the method below.**
 1. Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
 2. Wipe surface of the synthetic leather with the soft cloth.
 3. Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
 4. Wipe with a piece of wrung soft cloth.
 5. Have the synthetic leather take natural air drying.
- **If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it wipe the product.**
- **Do not dry the surface by a hair dryer**
- **Surface of the synthetic leather may be dyed by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.**
- **Keep these positions from long-time contact with plastics to avoid fading.**
- **Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.**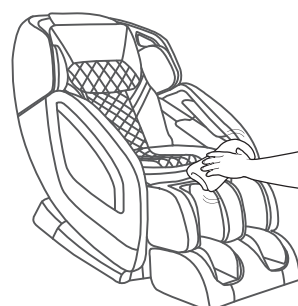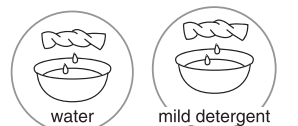

Cleaning of the plastic parts

- 1 Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.**
 - Do not clean the product with chemicals such as thinners, gasoline and alcohol.
- 2 Wipe with a piece of cloth once soaked in clean water and wrung out.**
 - Wring the cloth out before using it to wipe the remote controller.
- 3 Have the plastic parts take natural air drying.**

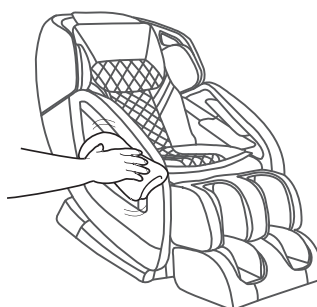Questions and answers (Q&A)

Q Can persons paying frequent visits to hospitals because of diseases use the product?

A These persons should consult physicians before using the product.
 Massage is the so-called "touch-pressure stimulation": Muscles are compressed to relieve their tension and promote hemokinesis. Some diseases may get worsened because of massage. Thus, consult the attending doctors before using the product.

Q How about the body height requirement for use of the product?

A The body height requirement differs with the specific body type. See the rough reference below when using the product.

If a person shorter than 145 centimeters or taller than 185 centimeters uses the upper body massage function and the Lower Leg/Foot massage function of the product at the same time, it will be possible that his soles will have poor contact with the foot massage section or that his knees will need to be bent.

For a taller person with knee bending when using the product:

Select method **A**

A Tilt the backrest and lower the Lower Leg/Foot massage section.

For a short person difficult to touch the foot massage section:

Select method **A**

A Raise the backrest and lift the Lower Leg/Foot massage section.

Q Are there any things needing special attention during moving?

A Pay attention to the following things during moving.

For convenience during moving, tilt backrest of the product till the lowest position.

During handling, do not sit on the product.

① Press ON button power on the product.

② Press **Backrest** to make the backrest completely tilt.

Press and hold on these two buttons until backrest and footrest stop

③ Turn off the power switch and pull out the power plug.


● If the original packing box is unavailable, make sure the power line is not wound on the product, wind it and place it on the seat surface. When handling the product, keep the power line from falling off and rubbing with ground.

● If the original packing box is unavailable, properly keep other packing auxiliary materials and wrap the product with them to avoid scratch and vibration during handling.

Q Where is serial number of the product?

A It is on back of the main body.

Serial Number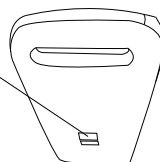

Q I care about the sounds the product makes when being used.

A It is caused by the core structure.

Q Does it matter if the massage wheels have different massage intensities?

A This phenomenon possibly exists during massage because of the core structure.

To simulate the most natural massage effect, the product is designed with a core structure allowing alternate back massage by a left massage wheel and a right massage wheel. During massage, the massage wheels possibly have slightly different positions on the back. That's why you have that feeling. It is not a fault.


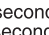
Q The acupressure force is too large.

A Set the air bag intensity to be "1" through the Air Bag button.
 You can also lower the Lower Leg/Foot massage section to reduce the acupressure force on the feet.

Troubleshooting

After a fault happens, inspect and try the corresponding method below.

If the fault still exists after that, contact the shop from which you bought the product.

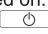
Fault	Cause and elimination method
The massage wheels stop during use (The product beeps and all icons and buttons on the remote controller flicker and then go out.)	<p><u>For safety, the massage wheels will possibly stop upon a too large external force during use.</u> If all marks and buttons on the remote controller begin to flicker, it suggests an extreme abnormality. In this case, turn off the power switch and turn it on about 10 seconds later. If the massage wheels stop again, adjust your sitting posture and do not lean on the backrest completely. (Users, particularly those heavier than 100 kilograms, should follow relevant precautions.)</p>
The product cannot tilt, or the footrest cannot be raised or lowered (The product beeps, the running light on the remote controller goes out and the product stops)	<p><u>For safety, the product will stop upon an obstruction or a too large force.</u> If all contents and buttons are flickering and the backrest or the footrest has stopped, turn off the power supply and turn it on about 10 seconds later and then operate the product.</p>
The massage wheels cannot reach the shoulders or the neck	<p>If the head is not in contact with the headrest, or the back is not in contact with the backrest, the shoulder position in body type detection will possibly be lower than the actual position. Sit in the deepest position of the product, lean your head on the headrest and operate from the beginning again.</p>
The left and right massage wheels have different heights (intensities)	<p>It is not a fault but a normal phenomenon because the product is provided with a massage core allowing alternate beats of the massage wheels.</p>
The product cannot run at all. ● Upper body (the neck to the waist) massage ● Lower leg/foot massage	<ul style="list-style-type: none"> ● The power line is disconnected. ● The power switch is not turned on. ● No program is selected after  on the remote controller is pressed.
The product cannot cool down even if the heating function is disabled.	<ul style="list-style-type: none"> ● Due to the heater structure, the product will still feel warm shortly after heating because of the residual heat. ● If a single part is continually massaged even after the heating function is disabled, you will feel increasingly warmer.
The power supply cannot be connected even if  is pressed again after the massage	<ul style="list-style-type: none"> ● The program will terminate about 5 seconds after the massage has ended. Press  at least 5 seconds after all the movements have stopped.
The product is damaged	<p>To avoid accidents, contact an authorized service center.</p>
The power line or the power plug has abnormal heating	

Troubleshooting

The sounds and feelings you care about in use of the product

You will hear or have the following sounds or feelings when using the product. It is not abnormal phenomena but results of the product structure.

- Sound when the stretching massages are retracted
- Sound and feeling when the massage wheels rub patterns on the fabric
- Sound of movements such as kneading and beat
- Friction sound of the running massage wheels
- Sound of up-down movement
- Sound of the conveyor belt
- Motor sound
- Sound of the squeezing massage wheels when they begin to stretch
- Sound when a person is sitting on the seat
- Friction sound between the armrests and the back cushion when the backrest is tilting
- Friction sound between the massage wheels and the fabric
- Sound of the inflator pump during Air Bag massage
- Sound of air exhaust during Air Bag massage
- Running sound of the solenoid valve at bottom of the product during Air Bag massage.
- Swing of the footrest
- Sound when the shoulders are stretched or moved
- Running sound of the air bags
- Sliding sound of the foot massage section
- Friction sound between the back cushion and the back cover leather when the backrest is tilting

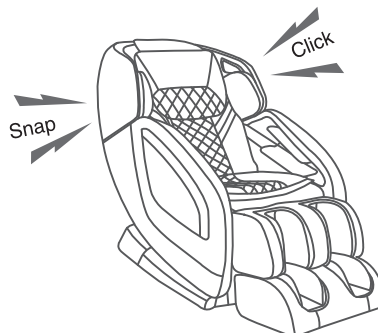If the sound is loud, it is also possible that the product has failed.

- In this case, have the product inspected and repaired by an authorized service center.

Warning

- **Do not refit, disassemble or repair the product without permission.** Otherwise fires, abnormal movements or personal injuries may happen.

Warning

- **In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid fuming, fires and electric shocks.**
 - < Abnormality and fault examples >
 - The product fails to work after the power switch is turned to the ON position;
 - After the power line is moved, the product is discontinuously powered on;
 - In use, the product gives off a burning smell or an abnormal sound;
 - The product is deformed or abnormally heating
- In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.

Product Specifications

Power supply		AC220-240V~,50/60Hz
Power consumption		120W
Upper body massage	Range of massage (up-down direction)	Approx. 97cm
	Massage speed	Kneading:Approx.21 times/min-Approx.37 times/min
		Beat:Approx.422 times/min-Approx. 703 times/min
		Moving speed:Approx.43mm/s
Adjust angle		Backrest: Approx. 124°~157° Footrest: Approx. 92.5°~ 115°
Automatic power-off		Approx. 15 minutes(automatic power-off)
Dimension	When it is not tilting (L x W x H)	Approx. 1420mm x 760mm x 1150mm
	When it is tilting (L x W x H)	Approx. 1640mm x 760mm x 935 mm
Weight		Approx. 78 Kg
Dimensions of packing box (L x W x H)		1180x770x1150mm
Weight(including the packing box)		Approx. 93 Kg
Operating Conditions		Ambient temperature:Approx.0°C~35°C Relative humidity:Approx.20%~80%
Storage Conditions		Ambient temperature:Approx.-5°C~35°C Relative humidity:Approx.20%~80%
Maximum body weight of user		Approx. 120Kg

For a user heavier than 100 kilograms, the product may have bigger noise and its fabric housing may be worn quicker.

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HONG KONG, CHINA
OGAWA Health Care International (HK) Ltd
31/F,YHC Tower,1Sheung Yuet Road
Kowloon Bay
Kowloon
Hong Kong
Tel:+852 3465 6262
Fax:+852 2410 0119

SINGAPORE
OGAWA Health-Care Pte Ltd
27 Tampines Industrial Avenue 5,
T5 @ Tampines
Singapore 528623

Tel:+065-6846 8588
Fax:+065-6846 7677

VIETNAM
OGAWA Vietnam Co Ltd
2nd floor,Petroland Tower,
12 Tan Trao,Tan Phu ward
District 7, Ho Chi Minh
Tel:+84 8 54133 222/54133 223
Fax:+84 8 5416 1736

PHILIPPINES
O-Healthcare Solution Phil. Inc.
2303, 4th Floor, G&A Building,
Chino Roces Avenue Extension,
Makati City, Philippines
Tel : +632 889 0676 / + 632 844 6851
Fax: +632 812 0421

Ogawa is grateful
to have received
these awards:

Ogawa Wellness Hubs : Malaysia . Singapore . Vietnam . Philippines . Indonesia . Myanmar . India . China .
Hong Kong . Taiwan . Turkey . Pakistan . Mauritius . Russia . France . Canada . Thailand .
Korea . Australia . Dubai . K.S.A . Nigeria . Egypt . Lebanon . Denmark . Sweden . U.S.A