# *SMARTEDGE*

Rev.1.0

User Manual  
OG 6338 使用手册

# *SMARTEDGE*

User Manual  
All information is correct at the time of print

# CONTENTS

Safety Instructions	03
Warranty Claims	05
Product Parts	06
Moving the Chair	07
Controller Diagram	08
Product Operation Guide	09
Product Features	14
Cleaning And Maintenance	15
Questions And Answers(Q&A)	16
Troubleshooting	17
Product Specification	19
Contact Us	20

# Safety Instructions

## Warning!

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in the risk of electric shock.

Check with a qualified electrician and service personnel if you are in doubt as to whether the product is properly grounded. Do not modify the product's accompanying plug. If the outlet is not suitable, be sure to have a proper outlet installed by a qualified electrician.

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire, or injury.

Read the following details before use:

1. Be sure to insert the massage chair plug into the proper electrical outlet.
2. Make sure that the power plug is securely inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use and before cleaning.
4. DO NOT use the massage chair in places where the humidity is high.
5. DO NOT use the massage chair outdoors and avoid places with direct sunlight.
6. If the power goes off during use, be sure to turn the power switch to the OFF position and pull the power plug out from the socket to prevent any damages when the power comes back.
7. Never operate the massage chair if the power cord is wet or damaged.
8. Never plug/unplug the power cord from the socket with wet hands.
9. Close and continuous supervision is necessary when this appliance is used by or near children, invalids or disabled persons.
10. Use this appliance only as described in this user manual.
11. Do not use any attachments other than those recommended by OGAWA.
12. Do not stand, sit or jump in the calves rest when the massage chair is in use. This may result in an accident or injury.
13. Do not attempt to stand on the backrest and armrest .This may result in an accident or injury.
14. Be careful not to fall asleep during the massage.

# Safety Instructions



15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair.
16. Do not remove the massage chair by pulling the power supply cord.
17. Be sure to check that there is no one (children in particular), pets or obstacles under the massage chair prior to lowering the leg rest /backrest.
18. Always check whether the surface of the massage chair is torn or ripped before use. If you find any unfamiliar tears in any area of the cloth in the backrest, do not use the massage chair. Kindly contact OGAWA service center immediately for repairs.
19. This appliance is designed for household use only.
20. For those suffering from illnesses, please consult your doctor on whether you can use the massage chair if in doubt.
21. The following persons are requested to consult a doctor prior to use in order to reduce the risk of the user feeling sick or suffering from an injury:
  - Those who have a malignant tumor;
  - Those who suffer from heart disease;
  - Those who are experiencing pregnancy, menstruation and puerperal;
  - Those who require complete rest;
  - Those who are under medical treatment;
  - Those who suffer from back problems or whose backbone is abnormal or bent.
22. If you experience pain during use, stop the massage chair immediately. If any injuries happen, consult a doctor immediately. Never massage on swollen or inflamed parts of the body.
23. This product is a non-professional appliance designed to provide a comfortable massage. Do not use this product as a substitute for medical treatment.
24. Do not use this massage chair for more than 30 minutes at one time and kindly limit the massage to a concentrated area for not more than 15 minutes. Over-massaging can result in overheating, and thus reducing the product's lifespan.
25. Do not operate where aerosol (spray) products are used or where oxygen is being administered.
26. The massage chair is not suitable for children aged 12 years old and below.

- 
27. If the power plug is damaged, it must be replaced by a special plug available from OGAWA or authorized agents.
  28. As a safety precaution , Motor will stop working automatically (for protection) if apply too strong external force during use. Once appear above, switch OFF power ,and switch ON again after 10 seconds . Please pay attention to this ,especially for user whose weight over 100KG.

## Warranty Claims



- 
1. Warranty is not transferable.
  2. This warranty is effective only if the product is purchased from OGAWA.
  3. Proof of purchase (original receipt) is required for all warranty repairs.
  4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited to the duration stated in the warranty policy.
  5. Damage due to misuse, improper treatment and unauthorized modification and repair are not covered by this warranty.
  6. Warranty does not cover accessories and attachments which do not belong to the massage unit.
  7. Warranty is not effective to rental, business, commercial, institutional or other non-residential users.
  8. All services covered by this warranty must be approved by OGAWA and repaired by authorized technicians only.
  9. If replacement parts for defective materials are not available, OGAWA reserves the right to make substitution in lieu of repair or replacement.
- 

# Product Parts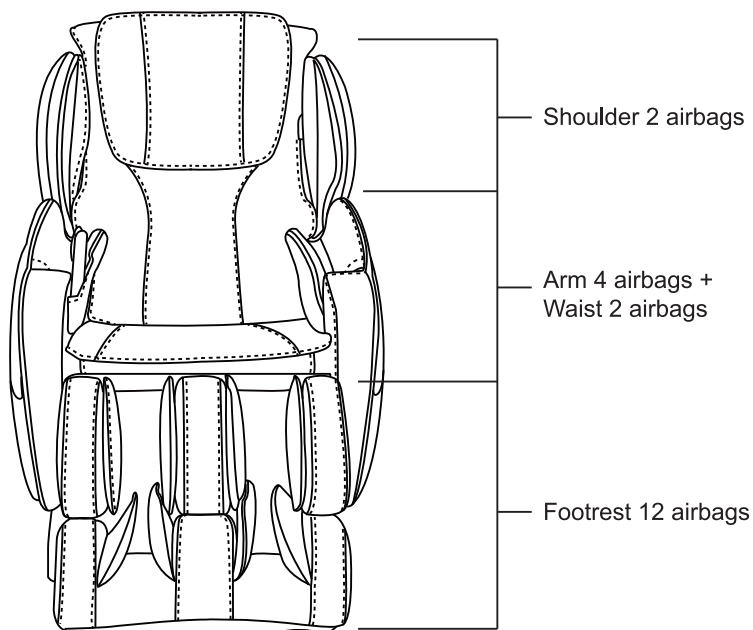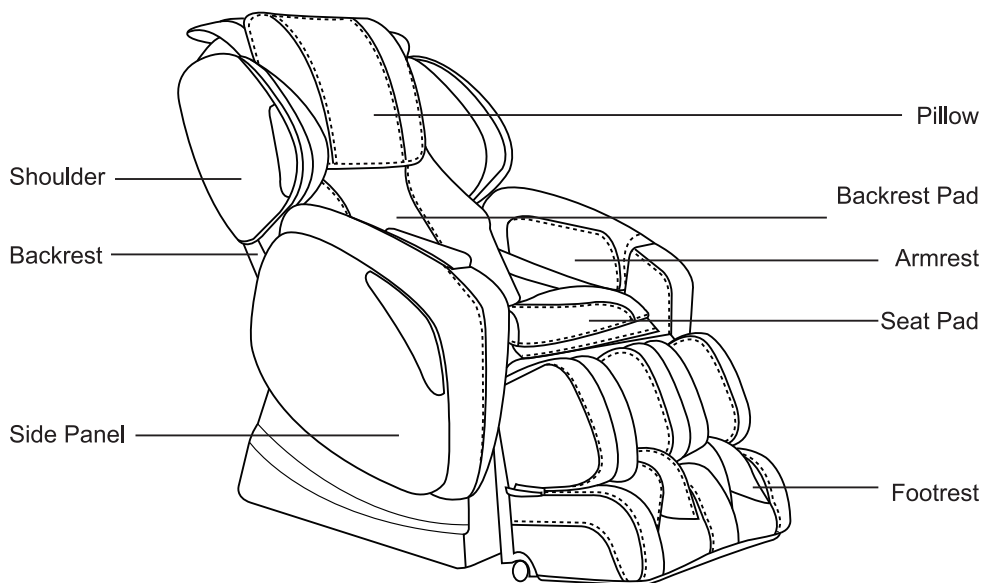

# Moving the Chair

Lift up the calves-rest, before reclining the chair to approximate 45° to easily move the chair around the house with a slight pull or push.

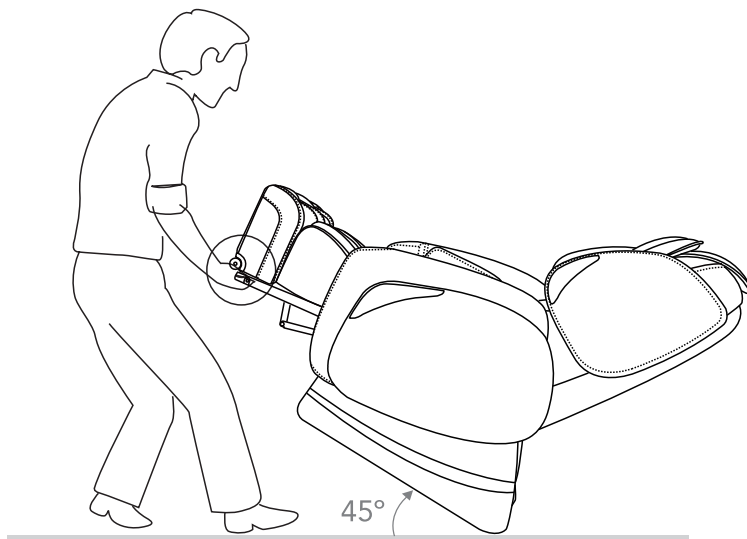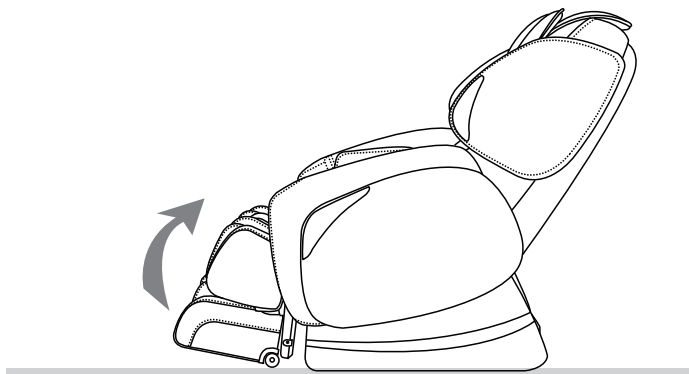
## Caution:

- i. Before moving the chair, please put the footrest back in the original position. Otherwise, injuries might occur in the process of moving the chair.
- ii. Avoid moving the massage chair when there is someone seated on it.
- iii. Placing a mat underneath the massage chair is advisable, as the moving process might leave scratches on the floor surface.

# Controller Diagram

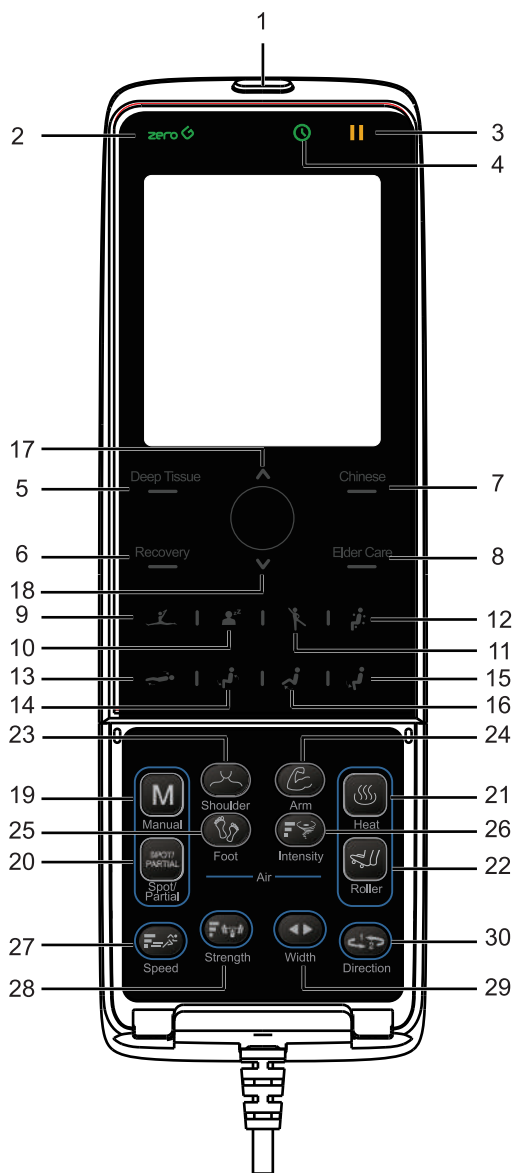# Product Operation Guide

Connect to the power supply, and turn the switch to: "I" position, and the chair is now in idle mode.

## Control Buttons for the Massage Chair

### 1 ON/OFF

#### The Start/Stop Control Button

Press once on this button, the LCD backlight will be lit. Press any massage function button to begin the chair's massage functions. This will also start the Timer. Press this button again to stop all massage functions, and the massage rollers, backrest, footrest will return to their original position.

### 2 ZERO-G

Press this button to activate the Zero-G function, and press again to return to the idle position.

Note : ZERO-G button is NOT controllable in the below message programs: 

### 3 PAUSE

#### The Pause Button

When the chair is in massage mode and the user wants to temporarily pause it, press the Pause button and all massage functions will stop. The "Pause" button will be blinking and the set time countdown will be paused as well. Press the Pause button again to immediately cease the "Pause" function and resume the original massage function. The countdown will continue.

### 4 TIMER

#### Time Interval Control Button

When a massage function is activated, the time is preset at 15 minutes. Every time this button is pressed, it will change the time interval sequence at "20>25>30>5>10>15>20" minutes.

# Product Operation Guide

## Full Body Auto Programmer

If either 1 of the 8 Auto Programs is activated, the chair will automatically perform a shoulder height scan. The user can manually adjust the shoulder height position to the desired comfort.

### 5. Deep Tissue

Mainly focus on shoulder and waist, massage using kneading and tapping for whole body relaxation.

### 6. Recovery

Deep massage on waist to create shaking buttock effect, massage using kneading and tapping, for body toxic release.

### 7. Chinese

Release stress, kidney care by using slow and soft kneading and hard tapping.

### 8. Elder Care

Slow down senescence, masage using soft clapping and kneading to release body pain, specially for senior.

Body stretchng by using airbag and roller.

Remark : Zero G, backrest and footrest adjustment is not available in this program.

Whole body massage, increase blood circulation, relaxation for sleep. Massage mainly on kneading and support by clapping.

Target on shoulder, waist and leg which the acid is accumulate especially after sport. Massage to release muscles stress and recovery.

Release tired by using air bag on shoulder, waist, arm and foot to massage.

## 13/14 INCLINE/RECLINE

### Angle Adjustment Control Button for Massage Chair

These two control buttons can adjust the Inclining/Reclining angle respectively. Press once on the Inclining or Reclining control button to adjust the Backrest and Footrest to a desired comfortable position.

# Product Operation Guide

## **15/16 INCLINE LEG/ RECLINE LEG**

### **Angle Adjustment Control Button for Footrest**

These two control buttons can adjust the massage angle on the Footrest. Press once on the Incline Leg or Recline Leg control button, and the Footrest will start to move; When the Footrest reaches the desired comfortable position, press the button again to stop it.

## **17/18 UPWARD/DOWNWARD**

### **The massage position adjustment button for spot and partial massage**

When the Spot or partial massage functions are activated, press and hold the button to move the massage roller upward or downward. Release the button when the desired position is achieved.

## **19 Manual Massage Function**

### **KNEADING, SWEDISH, TAPPING, SHIATSU, CLAPPING, ROLLING**

Press this button to activate the rhythmic kneading manual massage function for a full back massage at medium strength. Every time this button is pressed, the manual massage function will change in the sequence of KNEADING > SWEDISH > TAPPING > SHIATSU > CLAPPING > ROLLING > Stop.

## **20 SPOT/PARTIAL**

### **The Spot or Partial Back massage**

This button allows the user to select Spot massage for any part on the back. Press this button once to activate spot massage at the required location. Press this button again to switch to Partial function.

### **The partial Back Massage**

While running spot massage, when this button is pressed again, the massage will change from spot to partial. The massage will take the spot position as the centre to perform partial massage at a distance of approximate 8 cm. Press again to stop the function.

# Product Operation Guide

## 21 HEAT

### Setting HEAT

Press this button to activate the heating on waist.  
Press again to stop the function.

## 22 FOOT ROLL

### Foot roll function Start/Stop Control Button

Press once to activate the roller on foot, press again to stop the foot roll function.

## 23 Shoulder airbag

### Shoulder airbag control button

Press this button to activate shoulder airbag. Press again to stop this function.

## 24 Handrest airbag

### Handrest airbag control button

Press this button to activate handrest airbag. Press again to stop this function.

## 25 Footrest airbag

### Footrest airbag control button

Press this button to activate footrest airbag.  
Press again to stop this function.

## 26 INTENSITY

### The Intensity Adjustment Button for Air-pressure - 5 levels of intensity to be adjusted

After activating the Air-Pressure function (Auto and Manual), press this button to adjust the intensity in the cycle.

## 27 SPEED

### The Speed Adjustment Button for Swedish, Tapping, Shiatsu and Clapping - 5 levels of speed to be adjusted

After activating the Swedish, Tapping, Shiatsu, or Clapping function, every time the button is pressed, the speed can be adjusted to "Lv1> Lv2> Lv3> Lv4> Lv5" in one cycle.

# Product Operation Guide

## 28 STRENGTH

### **The Strength Adjustment Button for Kneading - 5 levels of strength to be adjusted**

After activating the Kneading function, every time the button is pressed, the strength can be adjusted to “ Lv1> Lv2> Lv3> Lv4> Lv5 “in one cycle.

## 29 WIDTH Adjustment

### **The Width Adjustment button for Tapping, Shiatsu, Clapping, Rolling**

After activating the Tapping, Shiatsu, Clapping or Rolling function, you can press this button to adjust massage width to “ narrow> medium> large “ in one cycle.

## 30 Direction of Manual Massage Function

When the manual massage function is operating, press this button to change the rhythmic mode of manual massage between 1 and 2.

## End Operation

Please press the “ON/OFF” button after use, and the chair will stop its operation, while the massage rollers will return to their original position; then switch the power supply button under the chair to “O” and unplug the machine from the electrical outlet.

# Product Features

OGAWA massage Chair is controlled by a microcomputer that performs various intelligent massage methods and combinations, like Kneading, Swedish, Tapping, Shiatsu, Clapping, Rolling, Footroller and Air Massage. All these massage methods accurately capture the acupressure points that could alleviate fatigue, promote metabolism and improve blood circulation effectively. Equipped with eighteen sets of full body automatic program, this product thoroughly fulfills your various needs.

Various types of manual / auto massage to suit your needs.

Spring-styled massage that is specially structured for the back is designed according to the curve of the human body to provide a more effective and thorough massage.


1. Designed according to body curve, to provide better massage sensations.
2. The Timer Function can be adjusted according to your desired duration of the massage programmer.
3. Liquid Crystal Display controller for a user friendly operation.
4. Built in with power surge and automatic temperature sensor for safety.
5. The operation is controlled by an intelligent microprocessor for excellent performance and consistent quality.
6. The machine radiates heat on the surface, Do Not use it over sensitive skin areas or in poorly-ventilated areas.

# Cleaning And Maintenance

## Cleaning of the synthetic leather

Wipe the synthetic leather with a soft dry cloth.

- **Do not clean the product with chemicals such as thinners, gasoline and alcohol.**
- **When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.**
- **If the synthetic leather is very dirty, wipe it with the method below.**
  1. Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
  2. Wipe surface of the synthetic leather with the soft cloth.
  3. Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
  4. Wipe with a piece of wrung soft cloth.
  5. Have the synthetic leather take natural air drying.
- **If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it wipe the product.**
- **Do not dry the surface by a hair drite.**
- **Surface of the synthetic leather may be dyed by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.**
- **Keep these positions from long-time contact with plastics to avoid fading.**
- **Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.**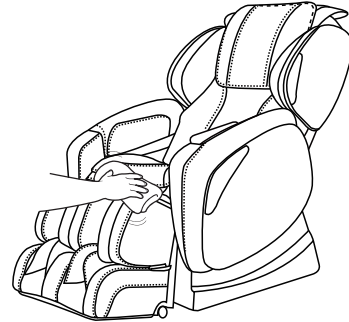

## Cleaning of the plastic parts


**1** Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.

**2** Wipe with a piece of cloth once soaked in clean water and wrung out.

- Wring the cloth out before using it to wipe the remote controller.

**3** Have the plastic parts take natural air drying.

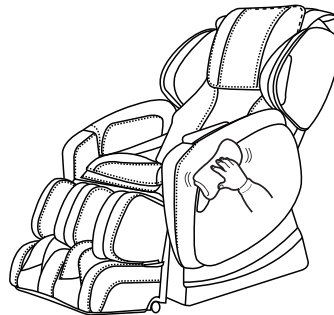# Questions and answers(Q&A)

**Q** Can persons paying frequent visits to hospitals because of diseases use the product?

**A** These persons should consult physicians before using the product.

Massage is the so-called "touch-pressure stimulation": Muscles are compressed to relieve their tension and promote hemokinesis. Some diseases may get worsened because of massage. Thus, consult the attending doctors before using the product.

**Q** How about the body height requirement for use of the product?

**A** The body height requirement differs with the specific body type. See the rough reference below when using the product.

If a person shorter than 145 centimeters or taller than 185 centimeters uses the upper body massage function and the Lower Leg/Foot massage function of the product at the same time, it will be possible that his soles will have poor contact with the foot massage section or that his knees will need to be bent.

For a taller person with knee bending when using the product:

Select method ①

① Tilt the backrest and lower the Lower Leg/Foot massage section.

For a short person difficult to touch the foot massage section:

Select method ②

② Raise the backrest and lift the Lower Leg/Foot massage section.

**Q** Are there any things needing special attention during moving?

**A** Pay attention to the following things during moving.

For convenience during moving, tilt backrest of the product till the lowest position.

During handling, do not sit on the product.

① Press ON button power on the product.

② Press  to make the backrest completely tilt.

Press and hold on these two buttons until backrest and footrest stop


③ Turn off the power switch and pull out the power plug.

● If the original packing box is unavailable, make sure the power line is not wound on the product, wind it and place it on the seat surface. When handling the product, keep the power line from falling off and rubbing with ground.

● If the original packing box is unavailable, properly keep other packing auxiliary materials and wrap the product with them to avoid scratch and vibration during handling.

**Q** Where is serial number of the product?

**A** It is on back of the main body.

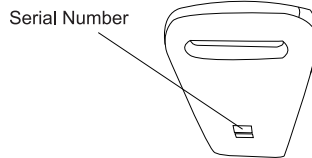**Q** What should I do if I do not feel warm in the waist?

**A** Make the waist in close contact with the warm transfer part. The warm transfer part is in close contact with the waist to make you feel warm. You may cover your waist with a blanket to get warmer.

**Q** I care about the sounds the product makes when being used.

**A** It is caused by the core structure.

**Q** Does it matter if the massage wheels have different massage intensities?

**A** This phenomenon possibly exists during massage because of the core structure.

To simulate the most natural massage effect, the product is designed with a core structure allowing alternate back massage by a left massage wheel and a right massage wheel. During massage, the massage wheels possibly have slightly different positions on the back. That's why you have that feeling. It is not a fault.

**Q** Theacupressure force is too large.

**A** Set the air bag intensity to be "1" through the Air Bag button. You can also lower the Lower Leg/Foot massage section to reduce the acupressure force on the feet.

# Troubleshooting

**After a fault happens, inspect and try the corresponding method below.**

If the fault still exists after that, contact the shop from which you bought the product.


Fault	Cause and elimination method
<p><b>The massage wheels stop during use</b> (The product beeps and all icons and buttons on the remote controller flicker and then go out.)</p>	<p><b><u>For safety, the massage wheels will possibly stop upon a too large external force during use.</u></b> If all marks and buttons on the remote controller begin to flicker, it suggests an extreme abnormality. In this case, turn off the power switch and turn it on about 10 seconds later. If the massage wheels stop again, adjust your sitting posture and do not lean on the backrest completely. (Users, particularly those heavier than 100 kilograms, should follow relevant precautions.)</p>
<p><b>The product cannot tilt, or the footrest cannot be raised or lowered</b> (The product beeps, the running light on the remote controller goes out and the product stops)</p>	<p><b><u>For safety, the product will stop upon an obstruction or a too large force.</u></b> If all contents and buttons are flickering and the backrest or the footrest has stopped, turn off the power supply and turn it on about 10 seconds later and then operate the product.</p>
<p><b>The massage wheels cannot reach the shoulders or the neck</b></p>	<p>If the head is not in contact with the headrest, or the back is not in contact with the backrest, the shoulder position in body type detection will possibly be lower than the actual position. Sit in the deepest position of the product, lean your head on the headrest and operate from the beginning again.</p>
<p><b>The left and right massage wheels have different heights (intensities)</b></p>	<p>It is not a fault but a normal phenomenon because the product is provided with a massage core allowing alternate beats of the massage wheels.</p>
<p><b>The product cannot run at all.</b> ● Upper body (the neck to the waist) massage ● Lower leg/foot massage</p>	<ul style="list-style-type: none"> <li>● The power line is disconnected.</li> <li>● The power switch is not turned on.</li> <li>● No program is selected after  on the remote controller is pressed.</li> </ul>
<p><b>No warm feeling in waist.</b></p>	<ul style="list-style-type: none"> <li>● The heating button is set to be OFF.</li> <li>● The warm feeling depends on factors such as body type, massage method, clothes and room temperature. Make the waist in close contact, the warm transfer part is in close contact with the waist to make you feel warm. You may cover your knees with a blanket to get warmer.</li> </ul>
<p><b>The product cannot cool down even if the heating function is disabled.</b></p>	<ul style="list-style-type: none"> <li>● Due to the heater structure, the product will still feel warm shortly after heating because of the residual heat.</li> <li>● If a single part is continually massaged even after the heating function is disabled, you will feel increasingly warmer.</li> </ul>
<p><b>The power supply cannot be connected even if  is pressed again after the massage</b></p>	<ul style="list-style-type: none"> <li>● The program will terminate about 5 seconds after the massage has ended. Press  at least 5 seconds after all the movements have stopped.</li> </ul>
<p><b>The product is damaged</b></p>	
<p><b>The power line or the power plug has abnormal heating</b></p>	<p>To avoid accidents, contact an authorized service center.</p>

# Troubleshooting

## The sounds and feelings you care about in use of the product

**You will hear or have the following sounds or feelings when using the product. It is not abnormal phenomena but results of the product structure.**

- Sound when the stretching massages are retracted
- Sound and feeling when the massage wheels rub patterns on the fabric
- Sound of movements such as kneading and beat
- Friction sound of the running massage wheels
- Sound of up-down movement
- Sound of the conveyor belt
- Motor sound
- Sound of the squeezing massage wheels when they begin to stretch
- Sound when a person is sitting on the seat
- Friction sound between the armrests and the back cushion when the backrest is tilting
- Friction sound between the massage wheels and the fabric
- Sound of the inflator pump during Air Bag massage
- Sound of air exhaust during Air Bag massage
- Running sound of the solenoid valve at bottom of the product during Air Bag massage.
- Swing of the footrest
- Sound when the shoulders are stretched or moved
- Running sound of the air bags
- Sliding sound of the foot massage section
- Friction sound between the back cushion and the back cover leather when the backrest is tilting

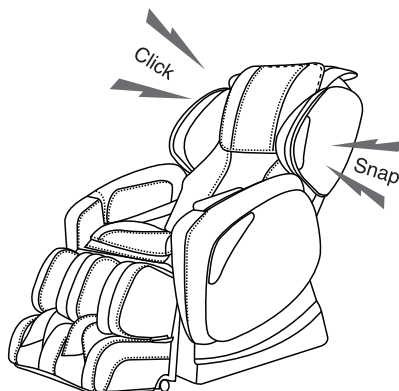**If the sound is loud, it is also possible that the product has failed.**

- In this case, have the product inspected and repaired by an authorized service center.

### **Warning**

- **Do not refit, disassemble or repair the product without permission.** Otherwise fires, abnormal movements or personal injuries may happen.

### **Warning**

- **In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid fuming, fires and electric shocks.**  
< Abnormality and fault examples >
  - The product fails to work after the power switch is turned to the ON position;
  - After the power line is moved, the product is discontinuously powered on;
  - In use, the product gives off a burning smell or an abnormal sound;
  - The product is deformed or abnormally heating  
→ In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.

# Product Specifications

Product Name	OGAWA SMART EDGE
Model number of the product	OG 6338
Power source	220V - 240V
Alternating current	50/60Hz
Power consumption	250W
Timer	5/10/15/20/25/30 minutes
Maximum operating period	30 minutes
Dimension (Length/width/height)	Upright position : 161 X 78 X 120 (cm) Reclined position : 198 x 78 x 86 (cm)
Weight	Product gross weight: 127 kg
	Product net weight: 106 kg
Length of the wire	Controller wire: 150 cm
	Power supply wire: 180 cm
Usage condition	Environment temperature: 10°-40°C
	Contrasting humidity: 30-85RH
Storage condition	Storage temperature: 20°-60°C
	Storage humidity level: 30-85RH
Safety feature	Equipped with overheated and power surge safety protection.

# Contact Us

**MALAYSIA**  
Healthy World Lifestyle Sdn Bhd  
No. 22 Jalan Anggerik Mokara 31/47  
Kota Kemuning, 40460 Shah Alam  
Selangor Darul Ehsan, Malaysia  
Tel : +603-5121 4286  
Fax: +603-5121 4386

**HONG KONG, CHINA**  
OGAWA Health Care International (HK) Ltd  
31/F,YHC Tower,1Sheung Yuet Road  
Kowloon Bay  
Kowloon  
Hong Kong  
Tel:+852 3465 6262  
Fax:+852 2410 0119

**SINGAPORE**  
OGAWA Health-Care Pte Ltd  
27 Tampine Industrial Avenue 5,  
T5 @ Tampines  
Singapore 528623  
  
Tel:+065-6846 8588  
Fax:+065-6846 7677

**VIETNAM**  
OGAWA Vietnam Co Ltd  
2nd floor,Petroland Tower,  
12 Tan Trao,Tan Phu ward  
District 7, Ho Chi Minh  
Tel:+84 8 3775 3093/3775 3094  
Fax:+84 8 3775 3095

**INDONESIA**  
PT Healthy World  
Lodan Center Block O No 20  
Jalan Lodan Raya No 2  
Jakarta Utara 14430  
Tel : +62 21-6983 1123  
Fax: +62 21-6983 1124

**PHILIPPINES**  
O-Healthcare Solution Phil. Inc.  
2303, 4th Floor,G&A Building,  
Chino Roces Avenue Extension,  
Makati City,Philippines  
Tel : +632 889 0676 / +632 844 6851  
Fax: +632 812 0421

# *SMARTEDGE*

User Manual  
All information is correct at the time of print

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O-Healthcare Solution Phil.Inc.