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CONTENTS

Thank you for choosing our products, please read this manual carefully and pay special attention to the safety precautions before use. Please keep it for future use.

Brief introduction	14
Graphene introduction	15
Infrared sensing	16
Target users	17
Packing list	18
Prepare before use	19
Product operations Guide	20
Functions	22
Troubleshooting	23
Maintenance	24
Ship&storage&specification	25

introduction

Product principle

The intelligent air pressure eye massage adopts the principle of traditional Chinese medicine and beauty treatment, combined with micro-chip control technology, improving eye blood circulation by hot compress, vibration and simulation of acupressure on the eyes. Built-in speaker with soothing music help to relax vibration and kneading.

1. Intelligent air pressure : vibration to acupuncture point of eye and eyelid, degree the eye and head nervous system
2. Hot compress with constant temperature: graphene as heating element, portable & constant temperature for eye massage, activates eye blood circulation, increases eye nutrient supply ability, effectively relieves eye dryness.
3. Vibration massage : vibrating the eye and eyelid, relieving blood circulation to reduce the elasticity of the ciliary muscle, enhancing the ability of the eye muscle to adjust.

Graphene

Graphene a planar film composed of carbon atoms. It is currently the world's lightest and most tough new nanomaterial due to its special nanostructure and high quality physicochemical properties. Graphene

Has important application prospects in the fvel of electronics, optics, magnetism, biomedicine.

Graphene Feature

portable convenient

stable performance


long life

Physiotherapy, with far-infrared ray, portable

Wave physiotherapist, most close to human body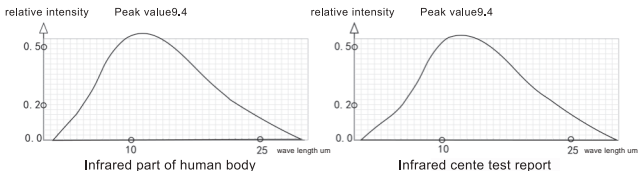

When graphene is heated, the far-infrared wavelength emitted is very close to the far-infrared wavelength of human body, both of which are 6-14um. When the two wavelengths interact equally, resonance occurs. When resonance occurs with cells, The far-infrared heat energy penetrates into the subcutaneous tissues, enables capillary expansion, promotes blood circulation, strengthens tissue metabolism, increases regenerative capacity, and improves immunity of the body, thereby achieving medical care.

Light of life - Infrared sensing

light of life power of health:

Far-infrared rays have been discovered as early as 1800 years ago. It is an electromagnetic wave that is invisible to the naked eye. It is inextricably linked to the life on the Earth. It is known that far infrared rays with a wavelength of 6-12 microns in sunlight are essential for human. The far-infrared rays emitted are of the same wavelength as that by human body, so that human body can absorb the light energy and exchange and transmit. Therefore, people call this long-range far infrared ray "the light of life".

Introduction to 10 categories of physiotherapy with far infrared rays

Target users

People who have myopia or hyperopia . peple with overused eyes or eyestrain or wear galsses all the time.

People with eye bags, black eyes or suffeehing from isomnia.

undergraduates, middle and primary school students, myopia

hyperopia

over used eyes or wear glasses all the time

Designer secretary Teacher

Driver

eyestrain workers

Computer operator

Insomnia

Long-term mobile phone player

Packing list

USB Cable

User manual

Storage bag

Prepare before use

- ▶ Please ensure the product is fully charged. With low battery the machine function is turned off or cannot be turned on, please charge it before using it. Be sure to unplug the USB cable after charging, to avoid charging for a long time.

Charging

1. Connect the USB cable to the product USB port.
2. The charging LED light is red during charging, and the indicator light turns blue when fully charged.
3. In the charging process, the product cannot work during charging (charging in the working state, the product starts the self-protection program, and automatically stops working).
4. The main unit is powered by a built-in rechargeable battery, and the charging time is 3-4 hours.
5. Prompt for low battery charge during use. Please pause the use and recharge the battery.

Multiple charging methods

charging methods	computer USB port
	mobile portable power bank
	power adaptor


working time

The default working time is 15 minutes. If you need to extend the usage time, you need to turn on power button again.

Product operation guide:

- 1.ON/OFF button: long press for 3 seconds to turn on, long press for 3 seconds to shut down,Connect to Bluetooth,Double click to play music,Double-click ON/OFF again to pause playing music.Short press to switch modes.
- 2.Play music with Bluetooth, speaker Reminder under connect status.
- 3.3D vibration mode-left side -right side- left&right side.
- 4.This product cannot adjust the volume, so set the volume.80% of the volume, suitable for most people, other people Adjust the volume level via your phone.
- 5.This Bluetooth name is IF-999.Please connect Bluetooth before playing music for the first time.
- 6.With low battery voice reminder function,when the battery voltage is low, the speaker will prompt "Battery low"


7.Infrared operation: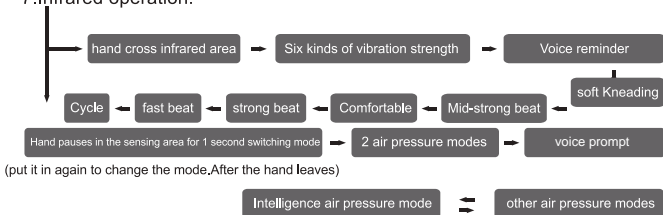

Attentions:

The six force function switches can only be switched in the vibration mode, otherwise it is invalid. Switching between the two air pressure modes only switches in the air pressure mode, otherwise it is invalid.

Function mode

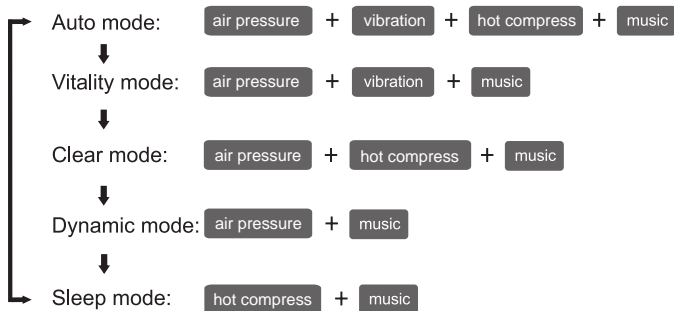Attentions

To ensure that you use the massager correctly, please refer to the precautions before use:

1. Do not use the eye massager while walking outdoors or driving.
2. Do not use the eye massager in a hot or humid place.
3. There are symptoms of eye diseases that cannot be used. Major eye patients such as eye surgery, glaucoma, cataract, and retinal detachment have been banned.
4. Patients with deep stroke, hypertension and hypotension should be used with caution. Please consult a professional doctor. If there is any discomfort during use, please stop using it immediately.

5. Persons with mental and physical discomfort or mental retardation should be used with caution. Please use it under the supervision of the accompanying staff. If there is any discomfort during use, please stop using it immediately.
6. Please avoid using the eye massager device when drinking alcohol or physical discomfort.
7. Eye massager has a hot compress function, which must be taken care of for sensitive people.
8. Put the massage device on the eye area then turn it on again to avoid damage to the air bag.
9. The contact lenses, lenses, hairpins, hair accessories, etc. worn before use should be removed first.
10. This massage device is a personal health care product and cannot replace medicines and medical equipment.
11. If you feel pain or serious discomfort during use, please stop using it immediately and consult a physician.
12. This massager is designed as a home electronic health care product and is not intended for use in diagnostic, therapeutic and other commercial applications.
13. If cleaning is required, wipe it with a cotton cloth soaked with medical alcohol. Do not wash the product with water. Do not soak it in liquid.
14. Do not disassemble this product by yourself.
15. Please use the power adapter configured by our company and pay attention to electricity safety.
16. Please use this product for closed eyes. After use, the eyes will be blurred. After closing eyes for 1-3 minutes, the eyes will slowly clear.

Troubleshooting

basic fault	possible cause	troubleshooting method
failure to start up	low battery	fully charge in time
	auto-protect power off	hold it 5 seconds for charge, let it to be unlocked
	unable to charge	damage adapter change adapter
low massage strength	low battery	charge in time

Maintenance

- 1.Clean with a dry soft cloth if there is dirt on the surface.
- 2.Clean with a soft cloth dampened with medical alcohol if particularly dirty.
- 3.Make the product in a safe place after using.
- 4.Take measures to prevent moisture, anti-rat,anti-pollution.
- 5.Please ask a professional technician to repair it if troubleshooting . Do not disassemble it at will.

Ship and Storage and Specification

Atmospheric pressure: 86 kpa~106 kpa

Ambient temperature: -10°C~65°C

Relative humidity: ≤80% RH, no corrosive gas and well ventilated room.

Transportation: This product is equipped with simple anti-shock facilities, suitable for air, rail, road and ship transportation. It should avoid rain, snow splash, inversion and collision. When the storage period is more than 6 months, it should be checked out of the box and energized for half an hour. It cannot be close to the ground, walls and roof. The room should be well ventilated to avoid strong sunlight and corrosive gas erosion.

Product specification

Product Name:	Eye Massager
Rating voltage:	3.7V
Rating power:	4.5W
Input voltage:	5V/1A

Contact Us

Malaysia

Healthy World (2nd) Sdn Bhd
No. 22 Jalan Anggerik Mallau 21/4F
Kota Kemuning, 40160 Shah Alam
Selangor Darul Ehsan, Malaysia
Tel: +603-5121 4388
Fax: +603-5121 4388

HONG KONG, CHINA

OGAWA Health Care International (HK) Ltd
21/F, WTC Tower 1 Shuang Yuen Road
Kowloon Bay
Kowloon
Hong Kong
Tel: +852 2663 6862
Fax: +852 2430 0119

SINGAPORE

OGAWA Health-Care Pte Ltd
27 Tampines Industrial Avenue 3,
T507 Tampines
Singapore 528423
Tel: +65 6486 8888
Fax: +65 6486 7677

VIETNAM

OGAWA Vietnam Co Ltd
2nd floor Parkland Tower
12 Tonkin, Tau Phu ward
District 7
Ho Chi Minh
Tel: +84 54133 232/54133 233
Fax: +84 5416 1736

PHILIPPINES

O-Healthcare Solutions Phil Inc.
2112, 4th Floor, GBA Building,
Chino Roces Avenue Extension,
Makati City, Philippines
Tel: +632 899 0476 / +632 844 6888
Fax: +632 812 0421